

Washable and Reusable, Staytex tubular dressings are easy to apply and reapply. The unique weave of the Staytex tube firmly maintains the dressing in place yet is comfortable and latex free.

Anasept® and Staytex+™ are trademarks of Anacapa Technologies, Inc.  
Anasept® is manufactured by Anacapa Technologies, Inc.  
San Dimas, California  
Orders/Information: 800-489-2591  
[www.anacapa-tech.net](http://www.anacapa-tech.net)

Staytex™ and Staytex+™ are manufactured in the USA for Anacapa Technologies, Inc.

Keep wound dressings in place with Staytex™ Elastic Tubular Dressings



Staytex™



Staytex is available in a pre-cut length of 22" that will fit up to 24" circumference.  
Catalog # 6000TX, Case of 12



Staytex+ is available in a pre-cut length of 34" that will fit up to 36" circumference. Catalog # 6006TXP, Case of 6




Staytex+ also comes in a convenient dispenser package with 120" long roll that can be cut to any length.  
Catalog # 6010FTX, Case of 6


## DIRECTIONS FOR USE:




**1** Cleanse the open wound with Anasept® Antimicrobial Skin and Wound Cleanser.

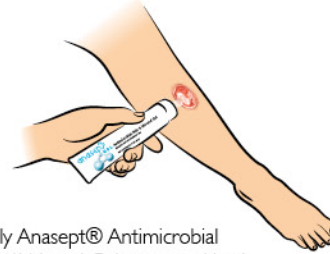


**2** Apply Anasept® Antimicrobial Skin and Wound Gel to wound bed.

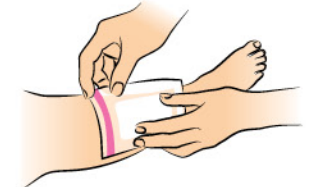


**3** Cover wound with sterile dressing.

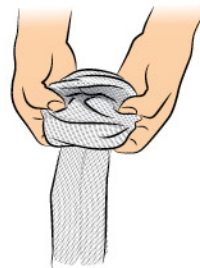

**4** Roll the Staytex™ tubular dressing up on your hands like you would a stocking before putting it on, and cut to needed length.

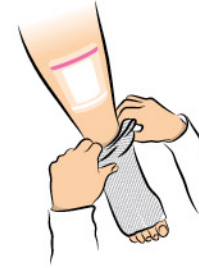

**5** Place Staytex™ over the foot until toes are exposed then unroll the rolled end up the length of the leg.

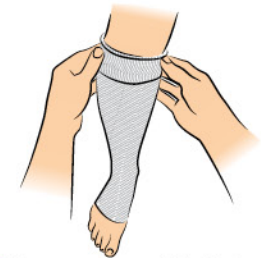

**6** To prevent Staytex™ tubular dressing from rolling down or up on the limb, turn down the top of Staytex™ making a three inch cuff.